- 1850CH** ARNESES
- 1852CH** المراجع
- 1854CH** ÚPONY
- 1856CH** HARNESS
- 1858CH** HARNAIS
- 1860CH** SISTEMI DI RITENUTA
- 1865CH** ARNESES
HAMURI
UCHYTENIE

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EN

Dear customer,

Thank you for trusting a Teyder product. You have purchased a quality product. Please, read the instructions carefully. If you have any doubt you can contact your doctor, orthopedist, or our customer service department.

We wish you have a quick recovery together with Teyder products.

QUALITY AND REGULATION

All our products have been manufactured following all CEE quality controls, and strictly following the European Regulation on Medical Devices – Regulation (EU) 2017/745 of the European Parliament of the Council.

SIZES

Selecting the correct size is important to guarantee the good product results. Please, consider the measurement charts presented on the product catalog to ensure that the size choice is correct.

INDICATIONS

Any restraint system is indicated to maintain the user's posture in the wheelchair, being the key to their quality of life. In general, the use of any of this range of products provides greater safety when moving, body stability and prevents slips and falls. Any of these products will be chosen according to the needs of postural control and support required by the user:

Harness: They are especially suitable for people with trunk instability. They allow complete support for the user, preventing the user from sliding off the seat (references 1850CH and 1852CH). In the case of the perineal harness (reference 1854CH), it is also indicated for patients with good anterolateral control with a tendency to slide off the seat.

Leg Abductor belt: These are the most suitable for users who need to keep both legs in permanent abduction (reference 1860CH).

Abdominal belt: Suitable for all types of people as they improve patient stability and safety (references 1856CH and 1858CH).

Wrist harness: Suitable for patients who need a complete support (reference 1865CH).

PRECAUTIONS

If while using the product you feel skin reaction, and allergy or skin irritation, please remove it quickly, and ask for medical advice.

Any serious incident involving the product should be reported to the manufacturer and to the competent authority of the member state in which the user and/or patient are established.

WASHING INSTRUCTIONS

Wash by hand with a neutral soap. Do not wash it with liquids containing alcohols, bleach or dissolvent neither do not dry clean. To dry the product, do not expose it directly to heat sources (direct sun light, radiators, heaters, etc.). Please make sure the product is completely dry before each use..

FITTING INSTRUCTIONS

1850CH- ZIPPED HARNESS

With the user seated, place the harness with the zipper closed around the abdomen, then pass the lower straps around the waist to the back of the couch or chair. Pass the straps over the shoulders until you reach the closure,

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PRIM, S.A.